

## TERMÉKUTASÍTÁS

HU



### ALPS PROSTHESIS-SOCK

KCN, KCM, KCW



ALPS prosthetic socks are light and soft, providing cushioning and comfort for an excellent fit. The socks are made with Coolmax fibers to create a moisture management system that wicks sweat away from the body through the fabric where it can evaporate quickly.

It adapts to changes in volume and can be worn with or without a prosthesis.

Color coded to identify thickness:

1 layer: Blue

3 layers: Yellow




5 layers: Green

## Function

The device is designed to be used as part of a lower extremity leg prosthesis.

Primarily used over a prosthetic liner to compensate for changes in residual limb volume; It also provides additional protection and cushioning of the skin and helps reducing friction between the prosthesis and the residual limb.

## Instructions for use and care

- Wear clean socks daily.
- Use only chlorine-free bleach. 
- Wash inside out.
- Tumble dry - delicate cycle, medium heat. 
- Machine washing is possible in warm water with gentle detergent 

## Warnings

Do not wear the socks on cracked or damaged skin

Do not continue to use if the device shows any holes or signs of deterioration.

## Guarantee

30-day satisfaction guarantee


The socks are guaranteed to be free of factory defects

### Choosing the right size

1. Measure the distance between the lower end of the residual limb and the top of the prosthetic socket. The length of the appropriate socks for this size is approx. 5 cm larger.

2. Measure the "bottom range" 5 cm above the bottom end.

3. Use the table below to match the measured data with the appropriate ALPS prosthetic sock size.

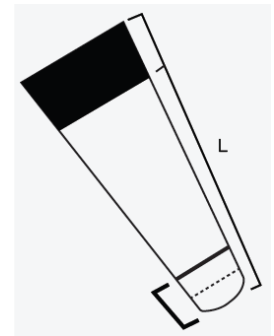

Size of the sock	Measured circumference of the stub	Length of the pod (min. - max.)	Description for the socks	Number of the items
08	20-23 cm	13-18 cm	Narrow extra short	KCN-08-(layer)(N/H/HR)
10	20-23 cm	18-23 cm	Narrow short	KCN-10-(layer)(N/H/HR)
14	20-23 cm	23-38 cm	Narrow normal	KCN-14-(layer)(N/H/HR)
18	20-23 cm	38-48 cm	Narrow long	KCN-18-(layer)(N/H/HR)
08	23-31 cm	13-18 cm	Medium extra short	KCM-08-(layer)(N/H/HR)
10	23-31 cm	18-23 cm	Medium short	KCM-10-(layer)(N/H/HR)
14	23-31 cm	23-38 cm	Medium normal	KCM-14-(layer)(N/H/HR)
18	23-31 cm	38-48 cm	Medium long	KCM-18-(layer)(N/H/HR)
08	31-36 cm	13-18 cm	Wide extra short	KCW-08-(layer)(N/H/HR)
10	31-36 cm	18-23 cm	Wide short	KCW-10-(layer)(N/H/HR)
14	31-36 cm	23-38 cm	Wide normal	KCW-14-(layer)(N/H/HR)
18	31-36 cm	38-48 cm	Wide long	KCW-18-(layer)(N/H/HR)

## List of symbols



Designed for a single user

## Disposal

The device and packaging must be disposed of according to the local or national environmental regulations.

Check local and national environmental regulations and dispose of them accordingly.



## EC Conformity

This product complies with the requirements of the European Regulation EU2017/745 for medical devices. This product is part of the regulation VIII. according to the classification rules described in its annex, it was classified as a class I asset.



## Reporting a serious incident

In an unlikely situation when a serious incident occurs with the device, it must be reported to the manufacturer and the competent national authority.



**ALPS South LLC  
ENGINEERING s.r.o.**

2895 42nd Ave. N  
St. Petersburg, FL 33714, USA  
Tel 1-727-528-8566  
Fax 1-727-528-5426  
[info@easyliner.com](mailto:info@easyliner.com)  
[www.easyliner.com](http://www.easyliner.com)



**CEpartner4U**

Esdoornlaan 13,  
3951DB Maarn,  
The Netherlands  
[www.cepartner4u.com](http://www.cepartner4u.com)



**ALPS CZECH**

Božkovské náměstí 17/21  
32600 Plzeň  
Czech Republic  
Tel : +420 377 223 127  
Fax : +420 377 223 010  
[info@easyliner.eu](mailto:info@easyliner.eu)  
[www.easyliner.eu](http://www.easyliner.eu)

5297 02811.6K6 A

2023. december